

**AnyHand®**



Disinfection and cleaning instructions



## 1 Introduction

This Disinfection Guide refers to the use of the AnyHand® in multiple patients in rehabilitation clinics, hospitals, therapy practices and other health care facilities, as well as the use of the AnyHand® at home.

This manual explains the AnyHand® disinfection and cleaning procedures recommended and validated by LIME medical GmbH.

- LIME medical GmbH gives no guarantee that deviations from the procedures listed below and the associated effects on product performance will be acceptable;
- Always follow the instructions of LIME medical GmbH when using the cleaning and disinfection procedures;
- **Risk of electric shock!** Never immerse the therapy device, the mains adapter and/or the mains cable in water. Always unplug the therapy device before cleaning or disinfecting it and make sure that it is completely dry before connecting it to the mains.

## 2 Parts suitable for Wipe disinfection

In principle, all externally accessible parts of the AnyHand® are suitable for wipe disinfection. Particularly the user interface, the finger mechanics, the finger holders and the silicone mats should be wiped disinfected between patient changes.

## 3 Preparing AnyHand® for cleaning/disinfection

Proceed as follows to correctly prepare the AnyHand® for cleaning and/or disinfection:

1. Make sure that there is no hand in the therapy device.
2. Disconnect the AnyHand® from the power supply.
3. Disconnect the USB connection of the therapy device to the Tablet/ PC.
4. Remove all finger holders from the finger mechanics. To do this, simply pull them out in the direction of the fingertip or opening.

## 4 Manually Cleaning

Before disinfection, the entire AnyHand® must be cleaned of visible dirt.

For cleaning, only use a cloth slightly moistened with water (room temperature, approx. 20 °C). Wipe the AnyHand® with the moistened cloth on the areas to be cleaned. Be careful to avoid damaging the AnyHand®. Then dry the AnyHand® with a dry, absorbent cloth.

The finger holders can be cleaned under warm (approx. 45 °C) running water.

**Do not use any cleaning agents. Under no circumstances apply mechanical cleaning procedures (such as dishwasher cleaning)!**

## 5 Manually Disinfection

Once you have cleaned the AnyHand® from any visible dirt, you can disinfect the AnyHand®.

Only use ethanol-based (70 - 94 %), aldehyde-free **wipe disinfectants** for disinfection.

Wipe the AnyHand® thoroughly with the wipe disinfection cloth. Be careful to avoid damaging the AnyHand®. Let the wiping disinfectant work as indicated on the wipe disinfectant container. Then let the AnyHand® air dry for five minutes.

**Under no circumstances use other disinfectants or spray disinfectants of any kind. Under no circumstances apply mechanical disinfection procedures (such as thermal disinfectors)!**

LIME medical GmbH recommends the use of **mikrozid® alcohol free wipes** (manufacturer: **Schülke & Mayr UK Ltd**).

**LIME medical GmbH accepts no liability for damage caused by the use of incorrect cleaning or disinfecting agents or incorrect cleaning or disinfection procedures!**

## Contact Details:

### Manufacturer:



Manufactured for:

**LIME medical GmbH**

Hauptstraße 17 + 19

Alte Waggonfabrik (Gebäude 6306)

55120 Mainz

Germany

Tel: +49 6131/2501053

E-Mail: [support@lime-medical.de](mailto:support@lime-medical.de)

Web: [www.lime-medical.de](http://www.lime-medical.de)

## Disinfection and cleaning instructions:

Revision US: 01

Language: English

Date: 2022-09-06